Impianto MITO Zigomatico

MITO Zigomatico Implant

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Juxta-sinusal zygomatic implantology, an evolution of the intra-sinusal technique with palatal access, allows simplification of the surgical procedure, reduction of intraoperative time, limitation of postoperative complications, and improvement of prosthetic function. In specific and well-defined clinical situations, the zygomatic implant becomes the only therapeutic solution for the patient.

Leader Medica has therefore developed the MITO Zygomatic system, which incorporates this innovation and includes a zygomatic implant with differentiated surfaces designed to simplify surgery, as well as a surgical kit specifically developed for the juxta-sinusal technique, containing diamond burs used to create the bone "slot" that guides the final drill. A complete prosthetic component range, including MUA abutments up to 60°, allows prosthetic management of all different implant emergence profiles.

The implant features a completely cylindrical morphology and is available in two diameters: 3.75 mm, suitable in cases of reduced zygomatic bone volume, and 4.20 mm, to be used when sufficient zygomatic bone is present. Leader Medica provides dental practitioners with a comprehensive theoretical and practical training program aimed at deepening knowledge of this surgical procedure and enabling its selection as a therapeutic indication in immediate-load implant-prosthetic rehabilitation of patients with severe maxillary atrophy.

Ø esagono 2,42 compatibile connessione Screw-Vent
Ø 2.42 hex, Screw-Vent-compatible connection

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A cylindrical zone of variable height with a smooth, machined surface. It is minimally invasive, with soft tissues in direct contact. Two diameters are available: Ø 3.75 mm and Ø 4.20 mm.

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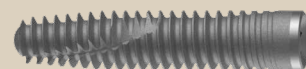
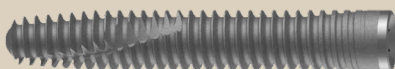
Threaded zone for anchorage in the zygomatic bone, 18 mm in height. Double acid-etched surface treatment, with a double-lead thread and a self-tapping profile. It allows easy and stable placement in the zygomatic bone.

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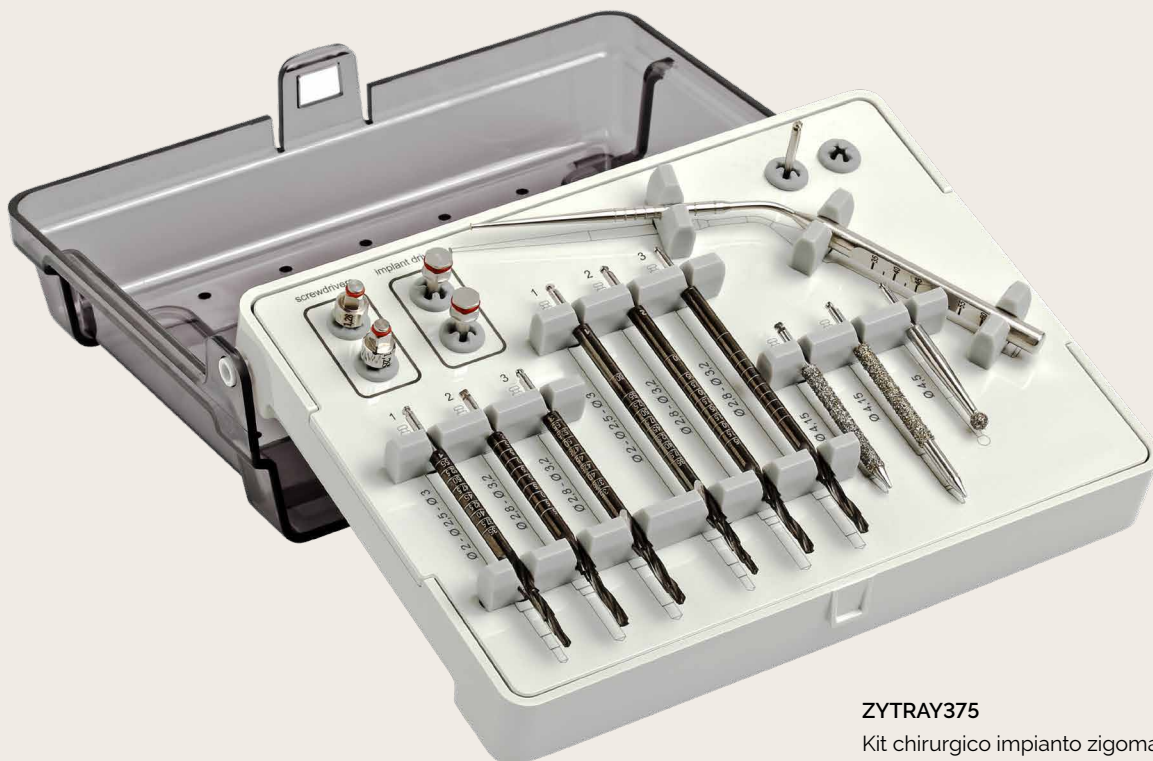
The MITO Zygomatic surgical kit, available for 3.75 mm and 4.20 mm diameters, is made of autoclavable and therefore sterilizable materials. The innovative drills included in the MITO Zygomatic surgical kit are the result of an extensive study carried out over the last four years by a team of clinical, scientific, and mechanical engineering consultants. This applies to both the steel drills and the diamond drills, whose surface treatments have been specifically developed for the MITO Zygomatic implant system. The drills allow implant site preparation through extremely precise and rapid consecutive steps, ensuring high effectiveness and, above all, strict compliance with protocol guidelines, thereby guaranteeing that the bone does not undergo any alteration or damage. The surgical kit includes all the surgical instruments required for the placement of zygomatic implants and prosthetic components. The manual screwdriver has been studied, designed, and manufactured by Leader Medica specifically for the MITO Zygomatic implant system, ensuring easy and high-performance use of the entire zygomatic system, even in the most complex surgical situations.

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MITO Zygomatic, Prosthetic Components

MUA – MONCONI MULTI UNIT - MULTI UNIT ABUTMENT

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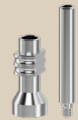
MUA - TRANSFER AND ANALOGUES

IL900003

Transfer pick up per MUA
Transfer pick up for MUA

TIS900001

Analogo per MUA
Analogue for MUA

Transfer pick up IL900003: vite IL900004 inclusa / *screw IL900004 included*

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Scanbody ILINTRA: vite IL100003 inclusa / *screw IL100003 included*

Analogo CAD / *CAD analogue* ILANCADMUA: vite TISANSCREW128 inclusa / *screw TISANSCREW128 included*

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Moncone provvisorio / *Temporary abutment* IL100222: vite IL100003 inclusa / *screw IL100003 included*

MUA - MONCONE CALCINABILE

MUA - CALCINABLE ABUTMENT

IL900002

Moncone calcinabile per MUA
Calcinable abutment for MUA

Moncone calcinabile / *Calcinable abutment* IL900002: vite IL100003 inclusa / *screw IL100003 included*

MUA T-BASE CROMO COBALTO CON CANNULA CALCINABILE

COBALT CHROME T-BASE MUA WITH CALCINABLE CANNULA

ILBACCR

T-Base in Cromo Cobalto con cannula calcinabile per MUA
Cobalt Chrome T-Base with calcinable cannula for MUA

T-Base MUA: vite IL100003 inclusa / *screw IL100003 included*

MUA T-BASE TITANIO

TITANIUM T-BASE MUA

ILBATI

T-Base in titanio per MUA
Titanium T-Base for MUA

ILBATISH

T-Base short in titanio per MUA
Titanium T-Base short for MUA

T-Base MUA: vite IL100003 inclusa / *screw IL100003 included*

VITI PROTESICHE PER MUA - MUA PROSTHETIC SCREW

IL100003

Vite protesica per MUA
Prosthetic screw for MUA

TIS100003TX

Vite protesica torx per MUA
Torx prosthetic screw for MUA

MITOVPR

Vite protesica per MUA angolato
Prosthetic screw for angled MUA

IL900004

Vite transfer per MUA
Transfer screw for MUA

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The advised prosthetic tightening torque is 25 N-cm

IL100003:

The advised prosthetic tightening torque is 15 N-cm